DK	GH Positionslås	2
GB/US	GH Positioning Lock	6
DE.....	GH Feststellbremse für Traverse und Hebemodul	11
SE.....	GH Positionslås	16
FR.....	GH Verrouillage Position Moteur	20
IT	Blocco di posizionamento GH	24
ES.....	Bloqueo de la grúa GH	28
BR	GH posição de trava	33

Manual – Version 3.00

Guldmann™

Vers. 3.00

Varenr.:

GH Positionslås 557600
 GH Positionslås - Single 557650

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1.00	Use of GH Positioning Lock	6
1.01	Manufacturer	6
1.02	Intended use	6
1.03	Conditions for use	6
1.04	Mounting a new GH Positioning Lock in an existing hoist/rail system	6
2.00	Description of features	7
2.01	IR remote control	7
3.00	Transport and storage	8
4.00	Maintenance and storage	8
4.01	Cleaning	8
4.02	Storage	8
4.03	How to prevent/avoid corrosion?	9
4.04	Daily maintenance	9
4.05	Disposal of GH Positioning lock, including batteries	9
5.00	Service and lifetime	9
5.01	Lifetime	9
5.02	Safety/service inspections	9
5.03	Accessories	9
6.00	Classification	9
7.00	EC-Declaration of conformity	9
8.00	Environmental policy statement - V. Guldmann A/S ..	9
WARRANTY – U.S.A and Countries outside the EU		
A.	Users guide	10
B.	WARRANTY	10

1.00 Use of GH Positioning Lock**1.01 Manufacturer**

V. Guldmann A/S
 Graham Bells Vej 21-23A
 DK-8200 Aarhus N
 Tel. + 45 8741 3100
 Fax + 45 8741 3131
 www.guldmann.com

1.02 Intended use

The GH Positioning Lock is used to lock a traverse rail system in a certain position. It is possible to lock a hoist and traverse rail individually or together at the same time.

The GH Positioning Lock is mechanical and is controlled via an infrared (IR) remote control. The system/hoist is placed in the desired position and then locked by pressing a button on the IR remote control. To unlock the system, press the remote control once again; see *section 2.00*.

1.03 Conditions for use

The use of the GH Positioning Lock is subject to the following prerequisites:

- The GH Positioning Lock is only designed for installation with the GH3+ ceiling hoist in A, B or C rails.
- The GH Positioning Lock may only be used by qualified personnel.
- The hoist may not be used to lift patients when one or more of the locks are activated.
- It is important that the lock is always disconnected before lifting patients (the user)
- The GH Positioning Lock may not be used with the following Guldmann products: GH Combi-Lock, GH Switch Track, GH Rail Curves GH Turntable, GH Drive motor for traverse rail or hoist with drive motor.
- When the positioning lock is activated and the lifting strap is angled more than 45 degrees from the vertical position, the hoist can be loaded with maximum 20 kg.

1.04 Mounting a new GH Positioning Lock in an existing hoist/rail system

The GH Positioning Lock is Class I equipment and must be installed by the Guldmann Service Team or by a certified installer.

Disconnect the equipment from the power supply by flipping the main breaker switch.

The GH Positioning Lock is equipped with batteries that require regular charging. The power supply for charging and the transformer must be connected by the Guldmann Service Team or a certified installer.

The GH Positioning Lock is charged automatically in one of the parallel rails. This ensures that the GH Positioning Lock will always work and be ready for use, and that the batteries last longer.

The transformer must be connected and switched on before charging can take place. A green indicator light on the transformer indicates that it is connected and turned on.

Always use the transformer supplied.

Safety concerning static electricity (ESD)

Service technicians and installers must use an ESD-safety package consisting of a mat, a ground wire, and a bracelet. The technician/installer connects the mat to a grounding point, for instance a radiator or a water pipe. The technician/installer must then put on the bracelet and connect it to the mat. If it is not possible to find a grounding point, the mat and the bracelet must be used as a minimum.

Only then is it allowed to work with the PC Board or components where it is possible to come into contact with the PC Board.

2.00 Description of features

2.01 IR remote control

The Positioning Lock is operated via IR remote control. Press the button once to activate the desired function.


The following functions can be activated by using the IR remote control for the GH Positioning Lock:


Button	Function	Movement
	Lifting strap up (If IR is installed in hoist)	
	Lifting strap down (If IR is installed in hoist)	
	Lock hoist	Only movement for traverse rail
	Unlock hoist	
	Lock traverse rail	Only movement for hoist
	Unlock traverse rail	
	Lock traverse rail and hoist	No movement
	Unlock traverse rail and hoist	
	Not given	
	Not given	

Label


The buttons' functions are also explained by the enclosed label (557599) which can be positioned as desired to serve as a guide for daily use. The symbols refer to the table above.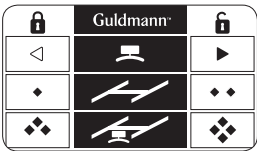


Lock hoist

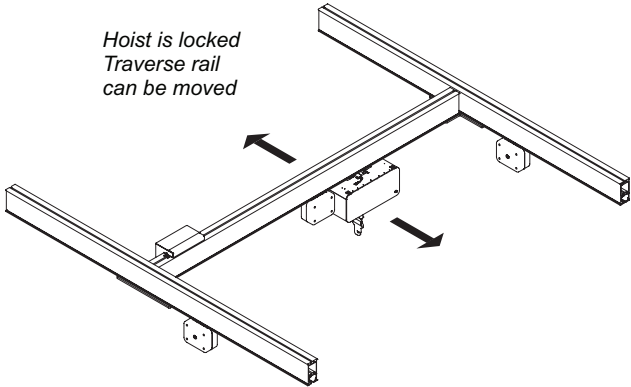Lock traverse rail

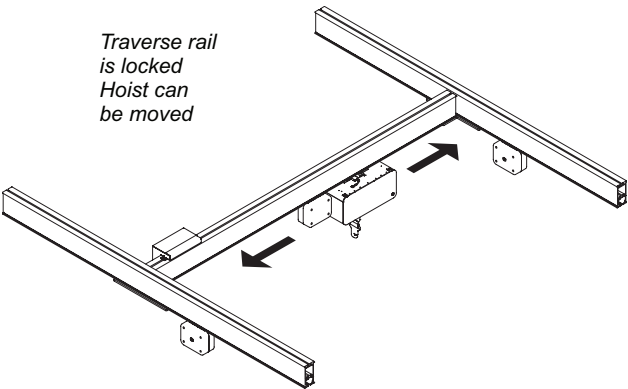
Lock hoist and traverse rail

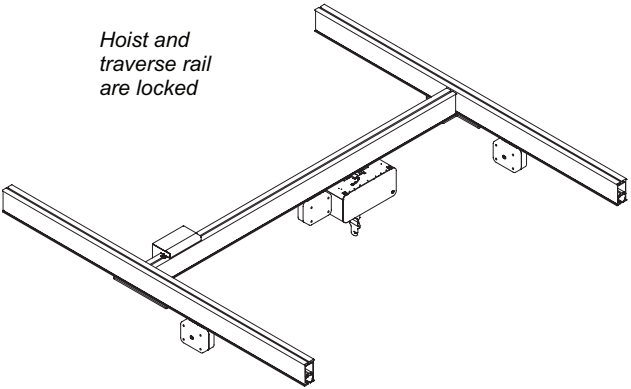3.00 Transport and storage

Guldmann recommends that the products are always transported and stored in the original packaging.

Key to symbols on the packaging:

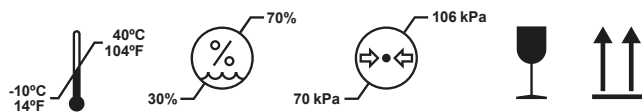The products should be stored at:

- Temperatures between -10 and +40°C / 14 and 104°F
- A relative air humidity of between 30 and 70 %
- An air pressure of between 700 and 1060 hPa
- This side up

4.00 Maintenance and storage

4.01 Cleaning

We recommend that the product and the parts that patients and caregivers can come in contact with are cleaned with Novadan Des Wipes or Novadan Desinfect O.

Desinfect O: A broad-spectrum disinfectant system as tablets. Suitable for disinfection of equipment, tools and surfaces in the healthcare sector.

Novadan Des Wipes: Is a disposable wipe for cleaning and disinfection of all types of surfaces, machines and instruments in the healthcare sector.

Note: Do not use strong acids, bases or alcohol to clean the products. Never clean the products in an autoclave.

Desinfect O is effective against:	Novadan Des Wipes is effective against:
<i>Microbial- & Virus test</i>	
• Staphylococcus aureus (ATCC 6538) (EN 13697:2001) • Enterococcus hirae (ATCC 10541) (EN 13697:2001) • Escherichia coli (ATCC 10536) (EN 13697:2001) • Pseudomonas aeruginosa (ATCC 15442) (EN 13697:2001) • Candida albicans (ATCC 10231) (EN 13697:2001) • Svinepest -Bovine viral diarrhea virus (BVDV) (EN 14675:2006) • Svine og fugle influenza (Influenza A virus H1N1) (EN 14675:2006)	• Staphylococcus aureus • Escherichia coli • Proteus mirabilis • Pseudomonas aeruginosa • Candida albicans • MRSA • H1N1 • HBV (HIV) • HCV

Novadan product	Description	Microbial and Virus test	Application	Dosing	Frequency
Desinfect O	A broad-spectrum disinfectant system as tablets.	Staphylococcus aureus, Enterococcus hirae, Escherichia coli, Pseudomonas aeruginosa, Candida albicans, Swine plague - Bovine viral diarrhea virus (BVDV), Swine and avian flu. (Influenza A virus H1N1)	Tools apparatus (Remember to wash afterwards)	1 tablet –10 l of water Time before effect: 5 min.	Daily/ periodically
Novadan Des Wipes	Is a disposable wipe for cleaning and disinfection of all types of surfaces, machines and instruments within the healthcare sector and food industry.	Staphylococcus aureus, Escherichia coli, Proteus mirabilis, Pseudomonas aeruginosa, Candida albicans, MRSA, H1N1, HBV (HIV), HCV	Lifting pool with adjustable handle, bed guard, remote control of beds, extension piece for bed guards, handle, polyester line for lifting pool.	Time before effect: 1 min. Apart from H1N1: 30 min	After every touching

4.02 Storage

See 3.00

For long-term storage, disconnect the battery plugs and the plug from the battery at the charging PCB.

4.03 How to prevent/avoid corrosion?

When the products are mainly used in an corrosive environment, e.g. swimming pool, the products must be ordered with a special corrosion-preventive surface treatment.

4.04 Daily maintenance

Ensure that the GH Positioning Lock is intact. Do not use the GH Positioning Lock if it is damaged or defective. Instead, contact the Guldmann Service Team or a technician certified in accordance with Guldmann's indications.

4.05 Disposal of GH Positioning lock, including batteries

Local and national regulations on environmentally correct recycling must be observed. Batteries (type NiMH) must always be delivered to an approved recycling point.

5.00 Service and lifetime

5.01 Lifetime

The products have an expected lifetime of 15 years, on the condition of correct use and correct service inspections, see section 5.02.

Replacement of components

Replacement of batteries, PCBs and lifting straps must be performed by a qualified service technician or the Guldmann Service Team.

5.02 Safety/service inspections

In accordance with international standard EN/ISO 10535 "Hoist for the transfer of disabled persons – Requirements and test methods" an inspection should be performed on the hoist at least once a year. Guldmann recommends that regular safety/service inspection is performed at least once a year with regard to the pattern of usage.

Safety/service inspections of the products must be performed by a qualified service technician or the Guldmann Service Team. In connection with the purchase Guldmann may offer a service agreement for this inspection.

During the safety/service inspection a report must be prepared on what was checked and replaced. Parts that are worn or defective must be replaced with new parts from Guldmann. Spare parts drawings and documentation can be obtained from the manufacturer or supplier.

Documentation/checklist regarding safety/service inspection can be obtained from the manufacturer or supplier.

5.03 Accessories

Infrared remote control

Guldmann item number 935034

Batteries

NiMH battery 24V/2000mAh, Guldmann item number 550574

Transformer

Transformer, Class I, Guldmann item number 550200

6.00 Classification

 CE marking

Read the manual before use


Must not be disposed of as standard household waste, must be recycled. genbrug.

Class I equipment: Permanent installation with protective ground


The equipment is not suitable for use in the presence of flammable mixtures.

Examples of serial number label

GH Positioning Lock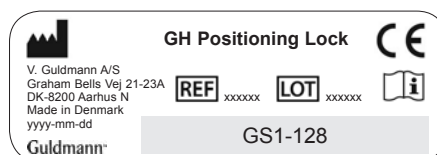

Transformer Class 1

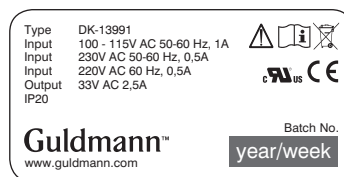7.00 EC-Declaration of conformity

The products are manufactured in compliance with the Council Directive 93/42/EEC of 14 June 1993 – with amendments, as medical device class 1.

8.00 Environmental policy statement - V. Guldmann A/S

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

WARRANTY – U.S.A and Countries outside the EU

A. Users guide

Before using the product, read the entire operation manual including warranty.

B. WARRANTY

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann designated technician. Any parts or components repaired or replaced by a Guldmann designated technician will be guaranteed for the remainder of the Warranty Period.

The warranty does not cover any part of the equipment which has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment which has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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Guldmann Srl
Guldmann Inc.